



**Manufacturer:**

MMN Medizin-Mechanik-Nord GmbH  
Russeer Weg 54a  
D-24111 Kiel  
Germany  
Tel. +49(0)431-6902814  
Fax: +49(0)431-6902815  
Mobil: +49(0)170-4446009  
E-mail: info@medizin-mechanik.de




**Appendix to Instructions for Use Version 2.0\_DE\_EN**

***Mounting Selfretaining Screwdriver with Quick Fix***


**1. Lock**



**2. Lock implant**

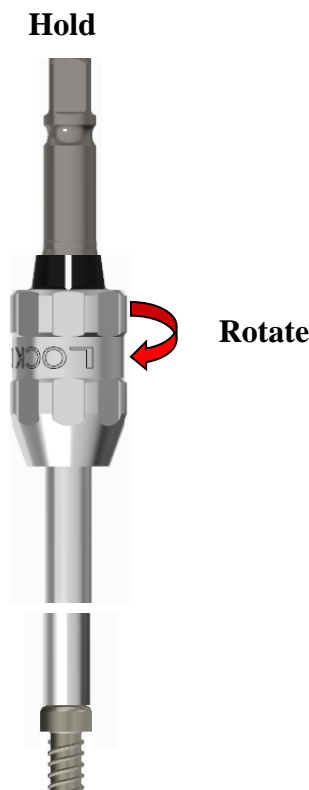

**3. Unlock**

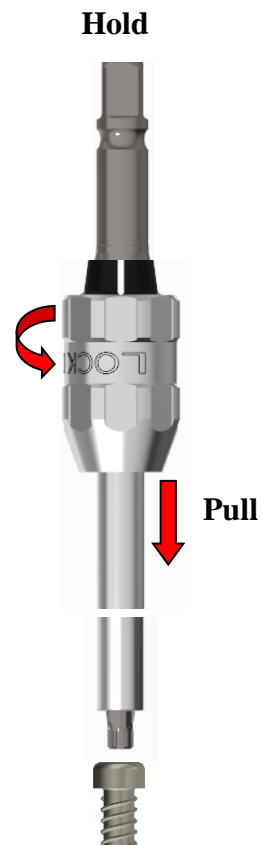

**OR-Nurse**

- Quick Fix with Screwdriver

**pre - assembled**

**Surgeon**

- Put on screw
- Rotate Quick Fix
- Screw locked

**ready for the surgery**

**Surgeon**

- Rotate Quick Fix
- Pull off Quick Fix

**ready for cleaning**



**Hersteller:**

MMN Medizin-Mechanik-Nord GmbH  
Russeer Weg 54a  
D-24111 Kiel  
Deutschland  
Tel. +49(0)431-6902814  
Fax: +49(0)431-6902815  
Mobil: +49(0)170-4446009  
E-Mail: info@medizin-mechanik.de



**Anhang zu Gebrauchsanweisung Version 2.0\_DE\_EN**

**Montage Selbsthaltender Schraubendreher mit Quick Fix**

**1. Verriegeln**




**OP-Schwester**

- Quick Fix mit Schraubendreher

**vormontiert**

**2. Implantat verriegeln**

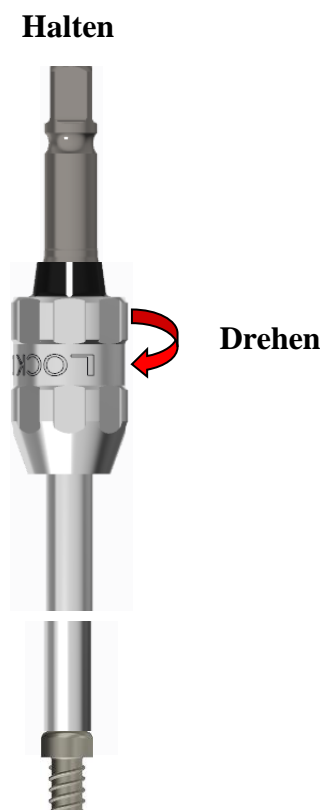


**Operateur**

- Schraube aufsetzen
- Quick Fix drehen
- Schraube verriegelt

**bereit für die OP**

**3. Entriegeln**

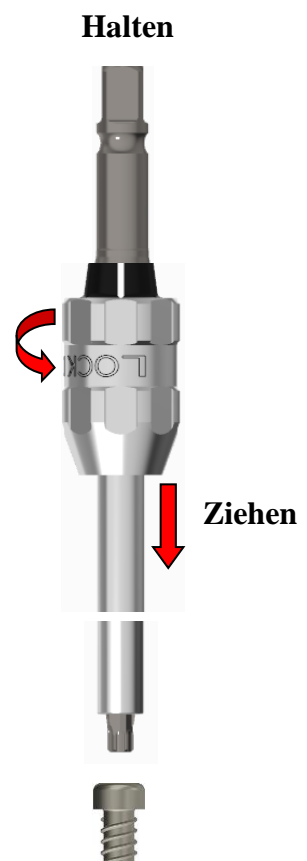

**Operateur**

- Quick Fix drehen
- Quick Fix abziehen

**bereit zur Reinigung**