## Manufacturer:

MMN Medizin-Mechanik-Nord GmbH  
 Russeer Weg 54a  
 D-24111 Kiel  
 Germany  
 Tel. +49(0)431-6902814  
 Fax: +49(0)431-6902815  
 Mobil: +49(0)170-4446009  
 E-mail: [info@medizin-mechanik.de](mailto:info@medizin-mechanik.de)

Version: 2.0\_DE\_EN

## Instructions for Use

### MMN Self-retaining Screwdriver Bit

#### Important information for doctors and OR staff

For detailed information concerning the identification of the product (such as associated implant, Cat. No.(REF)) please refer to the marking on the product and/or the labeling on the package. Ensure that you are familiar with the recommended uses, compatibility and correct handling of the Instrument; Important information - please read carefully prior to clinical use.

#### Available versions:

REF #	Benennung
25110	Self-retaining Screwdriver Bit Hexagon Socket 2,5 mm, AO
35110	Self-retaining Screwdriver Bit Hexagon Socket 3,5 mm, AO, Long
35111	Self-retaining Screwdriver Bit Hexagon Socket 3,5 mm, Jacobs Hall, short
35112	Self-retaining Screwdriver Bit Hexagon Socket 3,5 mm, AO, Extra short
35113	Self-retaining Screwdriver Bit Hexagon Socket 3,5 mm, Jacobs Hall, Long
35310	Self-retaining Screwdriver Bit T15, AO
35410	Self-retaining Screwdriver Bit T20, AO
35411	Self-retaining Screwdriver Bit T20, AO, increased Shaft Ø
35412	Self-retaining Screwdriver Bit T20, AO, short
35511	Self-retaining Screwdriver Bit T25, ¼", increased Shaft Ø
35512	Self-retaining Screwdriver Bit T25, AO, short
35610	Self-retaining Screwdriver Bit ISR20 ¼"
35611	Self-retaining Screwdriver Bit T25, ¼" cannulated
35720	Self-retaining Screwdriver Bit ISR25, ¼" increased Shaft Ø
35730	Self-retaining Screwdriver Bit ISR25 ¼" cannulated, increased Shaft Ø
35710	Self-retaining Screwdriver Bit ISR25 ¼"

#### Intended Use

The MMN Self-retaining Screwdriver Bits are intended for insertion and removal of bone screws in orthopedic and trauma surgery. The lockable screwdriver tip allows for a tight connection between screwdriver and screw. Duration of application: Short term application.

#### Handling instruction

The MMN Self-retaining Screwdriver Bits consist of two parts

- Sleeve with integrated locking nut
- Screwdriver shaft with screwdriver tip and coupling

For assembly of the device, the screwdriver shaft is inserted into the sleeve and turned counterclockwise until the left hand thread is fully covered by the sleeve. The screwdriver bit is now ready for use and is connected to a handle with coupling socket.

**Note:** The MMN Self-retaining Screwdriver Bits are intended for manual operation only and must not be connected to a power tool.

For assembly of the chosen bone screw to the Self-retaining Screwdriver Bit, the screwdriver tip is inserted into the respective socket of the bone screw and the locking nut is turned clockwise (LOCK ►) until the slip clutch of the locking nut is spinning. The slip clutch works a torque limiter, which allows for a reliable and tight assembly of the screwdriver in the respective screwdriver socket of the bone screw, but avoids damage of the tool by over-tightening of the locking nut.

**Note:** The *MMN* Self-retaining Screwdriver Bits shall be only assembled to bone screw sockets which have the same size as laser marked on the sleeve of the screwdriver bit (Hexagon 2.5 or 3.5, or Torx T15, T20, T25, ISR20 or ISR25). The screwdriver Bit ISR features a screwdriver blade that does not fit with wrench sockets having standard Torx. It is recommended to check the compatibility between Screwdriver Bit and wrench socket of the screw prior to use.

The bone screw is inserted with common surgical technique in accordance to the instructions given by the bone screw manufacturer. When the bone screw is in its intended final position the locking nut is turned counterclockwise (opposite to the designated direction of LOCK ►) which causes a disconnection of the screwdriver bit from the bone screw.

### Warnings and Precautions

The *MMN* Self-retaining Screwdriver Bits must be only used for their intended purpose.

It has to be considered that the devices are sensitive high-precision surgical instruments, which have to be handled with care. Improper use with application of unintended high forces (e.g. treatment with a mallet) or application of unintended high torque (i.e. when trying to disengage a cold-welded locking screw from a plate) may cause significant damage of the instrument.

Damaged or worn instruments must not be used and are to be discarded.

### Reprocessing

The *MMN* Self-retaining Screwdriver Bits are supplied in a non-sterile condition. Prior to first use and immediately after use the devices must be subjected to an appropriate cleaning and sterilization process. The number of reprocessing procedures is not limited.

**Cleaning:** The devices are multi-component instruments, which must be dismantled for adequate cleaning according to the assembly/disassembly instructions as described above in this document.

The following cleaning procedures are allowed for the *MMN* Self-retaining Screwdriver Bits: Automated cleaning, thermal disinfection.

The following Automated cleaning process parameters are validated by *MMN* and are recommended:

#### Manual pre-cleaning:

- immersion in lukewarm tap water for 5 minutes
- followed by brushing of the surfaces with a soft nylon brush under running tap water and cleaning of lumina with a suitable bottle brush until no residues are visible
- Treating in an ultrasonic bath for 5 min (frequency 35 kHz, room temperature) filled with 0.5 % cleaning solution (neodisher MediClean Dr. Weigert) in desalinated water. Before starting ultrasound, the lumen is to be filled with cleaning solution.
- Cleaning of lumina and gaps using a water jet gun for 15 seconds at 2 bar (30 psi).

#### Automated cleaning:

- Automated cleaning with following steps (prior to every step the instrument dishwasher has to be drained)
  - 2 min pre-cleaning in cold tap water (16°C ± 2° C) (61°F ± 3° F)
  - 5 min cleaning with warm tap water (55°C, 131°F) and 0,5 % neodisher MediClean (Dr. Weigert, Hamburg)
  - 3 min rinsing in lukewarm desalinated water (20°C ± 2°C) (68°F ± 3°F)
  - 2 min rinsing in lukewarm desalinated water (20°C ± 2°C) (68°F ± 3°F)
  - thermal disinfection taking into account the national requirements  $A_0 > 3000$  at 90°C (194°F) and at least 5 minutes holding time

**Maintaining:** After the cleaning procedure, the devices are dried and to be inspected for wear or damage and must be tested for proper function. Damaged or worn instruments must be sorted out for rework or replacement. Prior to sterilization, all hinges or mobile connections shall be lubricated with medical grade instrument oil that is explicitly suitable for steam sterilization.

**Sterilization:** The *MMN* Self-retaining Screwdriver Bits may be sterilized in assembled condition. The devices must be sterilized by means of a validated sterilization procedure before use. The following process parameters are validated by *MMN* and recommended for moist heat sterilization according to EN ISO 17655-1:

#### Method (outside USA): Moist Heat Sterilization

- Cycle: Saturated steam with forced air removal
- Temperature: 132–137°C (270–277°F)
- Exposure time: 4 to 18 Minutes
- Drying time: 30 minutes (recommended minimum, in chamber)

#### Method (USA): Moist-Heat Sterilization acc. to ANSI/AAMI ST79

- Cycle: Pre-Vacuum (Pre-Vac)
- Temperature: 270 °F (132 °C)
- Exposure time: 4 minutes
- Dry-Time: 30 minutes (minimum, in chamber)
- Cool-Time: 60 minutes (minimum, at room temperature)

**Note:** Not for prion inactivation.

**Note:** It is the responsibility of the healthcare establishment to refer to the sterilizer manufacturer's instruction for use and load configurations and sterilization accessories.

Symbol explanations:

					
Manufacturer	Catalogue Number	Charge-Number	Quantity	Non-sterile	Consult instructions for use

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