




Merit Ring


Quick User Guide

1. Connection, Connect and Disconnect

- A** Scan the QR code to download the Do Merit App (or download the Do Merit App in the app store or Google Play store).
- B** Place the smart ring on the charging accessory that is connected to power. The device will activate automatically (Ensure the ring is charged when you connect the ring).
- C** Open "Do Merit App," click "ring icon"  at the home page to enter the ring connection interface "Connect Ring," and search the ring's Bluetooth ID.
- D** When the ring appears in the app's Bluetooth device list, tap the listing to connect.
- E** The Ring and Do Merit App will automatically connect and pair. During connecting, the ring will flash a green light.
- F** If you need to disconnect the ring, click "ring icon" on the home page of the app. Click "Disconnect Ring".




Do Merit App

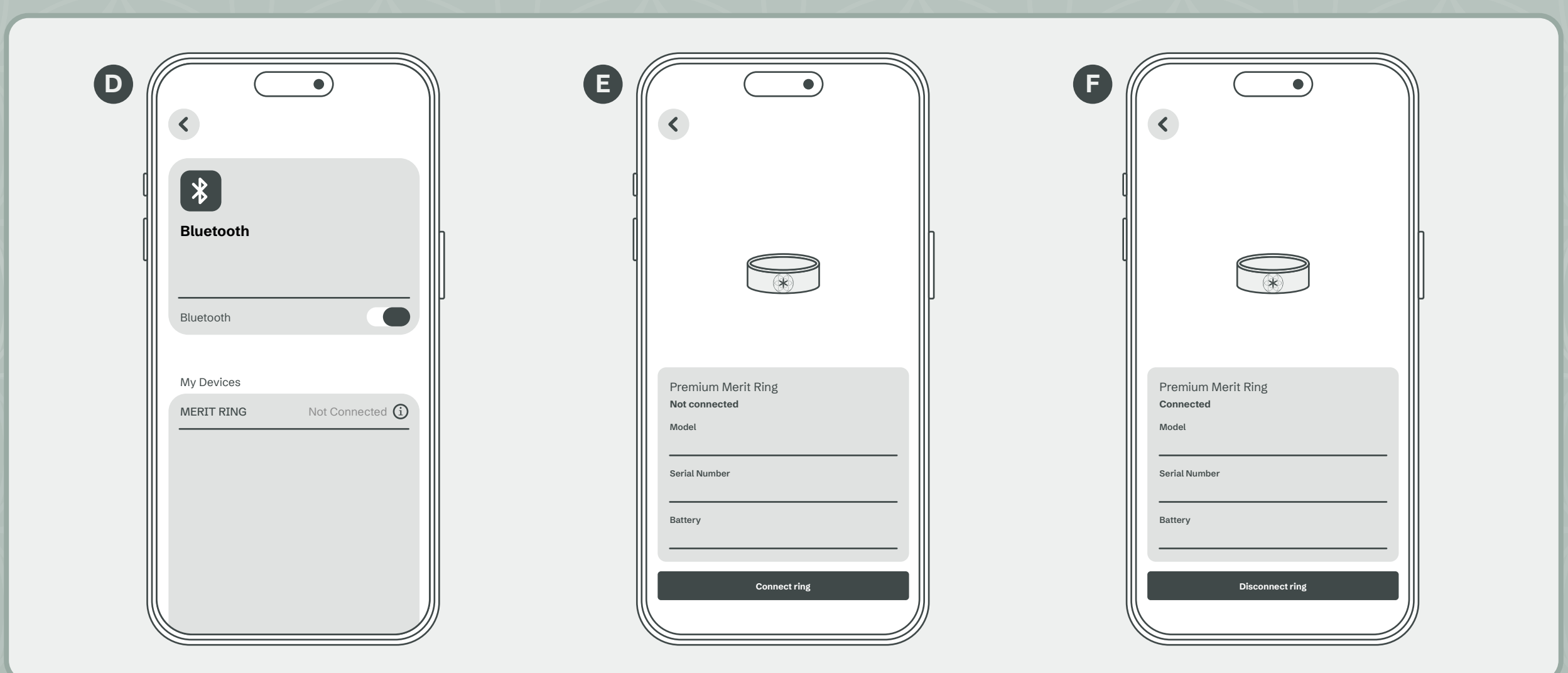

2. Wearing Method

Recommended to wear the ring with its sensors against the abdomen of your finger for more accurate measurements. To use the smart touch feature more easily, it is recommended to wear the ring on your index finger.

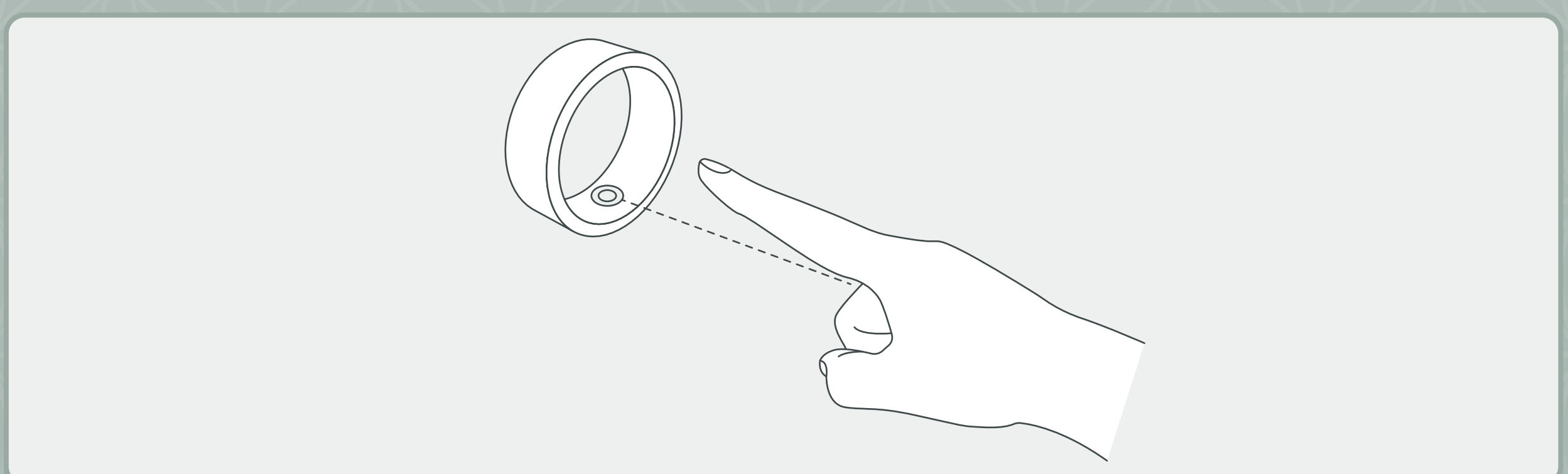

3. Charging

- A** Ensure the charging accessory is powered on. Place the ring on it and align it with the magnetic port. When charging, the red light will stay on continuously.
- B** When the battery is fully charged, the ring's green light will stay on continuously.

