### Instructions for Use

Non sterile Guide Wire Insertion Tool (PN 10610000) for sterilization and subsequent sterile single use.

**Intended Use**  
The Guide Wire Insertion Tool is intended to provide smooth passage of a guide wire through a hemostasis valve (Y-PTCA Connector) during a PTA/PTCA-procedure.

**Sterilization Instructions**  
To maintain the sterility the products must be packaged in a primary package according to EN 868 / EN ISO 11607. It is the responsibility of the processor that packaging is carried out under suitable conditions concerning deaeration. Therefore process validation and routine process monitoring are necessary.

The following EO sterilization process parameters have been validated:  
EO concentration: 800 mg/l  
Relative Humidity: 30 - 90 % RH  
Temperature: 48 - 58 °C  
Air removal: max. vacuum -60 kPa  
Conditioning: 10 min  
EO Injection: ≥ 30 min, max. pressure 40 kPa  
EO Exposure Time: 480 min  
EO Removal: -30 kPa  
Aeration: 5 air change cycles  
Flushing: 1 68 h at room temperature  
The sterilization instructions above are capable of ensuring the sterility of the product. It remains the responsibility of the processor to ensure that the sterilization under the processor-specific conditions, as actually performed, using equipment, materials, sterilize load, and product density in the processing facility, achieves the desired result. This requires validation and routine monitoring of the process.

### Instructions for Use

1. For use by physicians and healthcare professionals

**Discrimination:**  
Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**Instructions for Use:**  
1. The product may only be used in a sterile condition.  
2. Open the Y connector valve.  
3. Advance the guide wire insertion tool through the insertion tool until it has passed the haemostasis valve of the Y connector.  
4. Push the proximal end of the guide wire through the hemostasis valve of the Y connector.  
5. Withdraw the insertion tool from the distal end of the guide wire before starting the procedure.

**Precaution**  
1. Use proper aseptic techniques when handling. 2. There are no safety-related aspects to be considered during disposal.  
**Warnings**  
1. Do not use if the sterile packaging is damaged. 2. The product is intended for SINGLE USE ONLY. 3. Reuse of a single-use device creates a potential risk to the patient or user. This may lead to a contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death of the patient.  
4. The product should not be used for anything other than the intended use.

**Storage**  
Store in the original packaging, under clean and dry conditions.  
The above mentioned validated sterilization-conditions are suitable for the intended use.

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