Instructions for Use

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Storage
Store in the original packaging under clean and dry conditions.

Precaution
1. Use proper aseptic techniques when handling.
2. There are no safety-related aspects to be considered during disposal.

Warnings
1. Read any instructions before use.
2. Do not use if the sterile packaging is damaged.
3. The product is intended for SINGLE USE ONLY.
4. Reuse of a single-use device creates a potential risk to the patient or user. This may lead to a contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death of the patient.
5. The product should not be used for anything other than the intended use.

Intended Use
The Radial Compression Device is intended to assist in achieving haemostasis after removal of a needle, sheath or catheter from a vascular vessel during a PTA-/PTCA-procedure.
Non sterile Radial Compression Device (PN 10920X0) for sterilization and subsequent sterile single use.

Technical Description
18cm, 22cm or 26cm wristband.

Sterilization Instructions

To maintain the sterility the products must be packaged in a primary package according to EN 868 / EN ISO 11607. It is the responsibility of the processor that packaging is carried out under suitable conditions concerning cleanroom. Therefore process validation and routine process monitoring are necessary.
The following EO sterilization process parameters have been validated:
EO concentration: 800 mg/l
Relative Humidity: 30 - 90 % RH
Temperature: 48 - 58 °C
Air removal: max. vacuum -60 kPa
EO Injection: ≥ 30 min, max. pressure 40 kPa
EO Exposure Time: 480 min
EO Removal: -30 kPa
Flushing: 5 air change cycles
Aeration: 168 h at room temperature

The sterilization instructions above are capable of preparing the sterile product. It remains the responsibility of the processor to ensure that the sterilization under using equipment, materials, sterilizer load, and product density in the processing facility, achieve the desired result. This requires validation and routine monitoring of the process.
Note:
Compression pressures and compression times may differ according to the patient's condition, the heparin volume and the size of the puncture site.

1. The product may only be used in a sterile condition.
2. Open the package and remove the Compression Device.
3. Loosen the velcro tab and pass the Compression Device over the hand.
4. Align the centre of pressure plate with a gauze pad over the sheath access point, pull the fixing strap tight and secure using the velcro tab.
5. Tighten the pressure plate by turning the rotor to stop the immediate blood loss. Do not fasten the rotor too tightly.
6. While slowly removing the sheath tighten the pressure plate by turning the rotor until haemostasis is achieved. Take care not to over-tighten, it is advisable to retain some pulse.
Note:
The pressure indications on the side of the rotor are for convenience only.
Initial compression should be maintained for at least 30 minutes.
The Compression Bandage can be loosened every 15 to 30 minutes by turning the rotor until unassisted haemostasis is achieved.
9. Once full haemostasis is achieved, withdraw the plate back to the strap plate using the rotor and undo the velcro to remove the Compression Device completely.

Instructions for Use
1. For use by physicians and healthcare professionals

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