


Clinically Engineered Fixed Flow Rate Control

EMED's Infusets are fixed flow rate control infusion sets developed to be compatible with various mechanical infusion devices in the market. Infusets are rated to operate within pressures of 1-30 PSI and ensure usability with a wide variety of therapies.

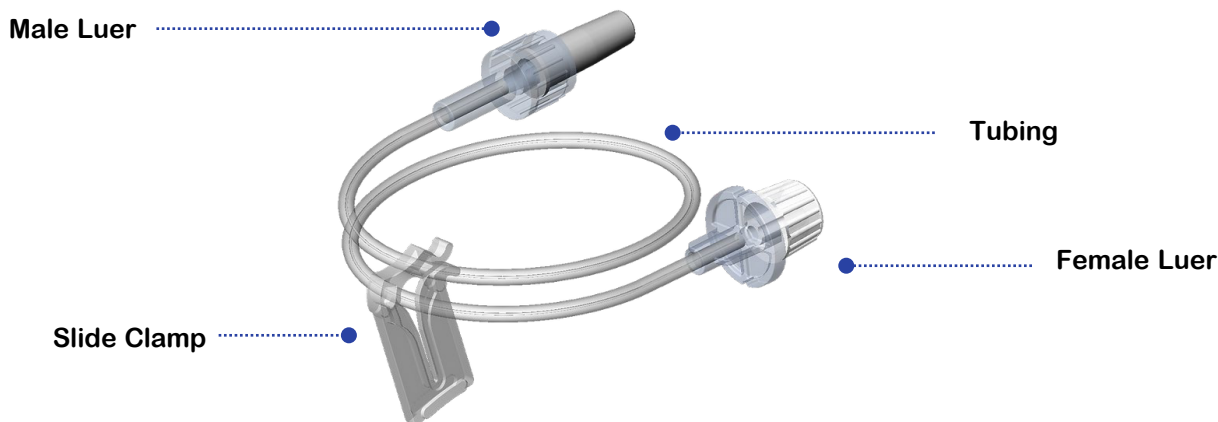

Features

Minimal priming volume


Pressures up to 30 psi

Highly flexible tubing

Universal luer fittings to fit standard syringes and needle sets

Optimized luer design with enhanced structural support (disk)

Translucent ergonomic clamp

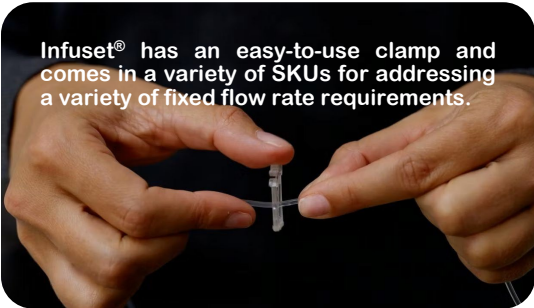

Infuset® has an easy-to-use clamp and comes in a variety of SKUs for addressing a variety of fixed flow rate requirements.

Benefits

Reduced waste of biologics

Simplified ambulatory use

Compatible with multiple biologics

Compatible with multiple mechanical pumps

Advantages

High precision. Accuracy levels above 90%

Manufactured with superior polymers to minimize chemical interaction with pharmaceuticals

Equipped with a male and female luer lock to connect to virtually any needle set and mechanical pump

Flow rate data charts for specific applications

Competitively priced, low single use cost

Quick Reference


See instructions for use for complete guide.




CONNECT syringe male luer lock (MLL) to Infuset female luer lock (FLL).



CONNECT Infuset male luer lock (MLL) to specified patient administration set female luer lock (FLL).

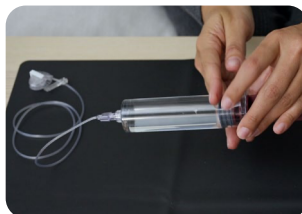

PRIME the tubing by carefully pushing on the syringe plunger to fill the tubing with immunoglobulin solution, or as instructed by your healthcare provider.

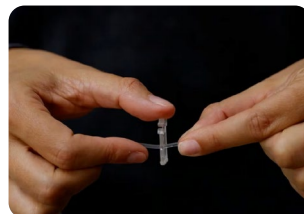

USE CLAMP provided with the Infuset to prevent the flow of immunoglobulin solution.

Ordering Information

For orders and inquiries, contact your local EMED representative or EMED's customer service department at 916-932-0071. Orders can also be placed via email at orders@emedtc.com.

EMED Reorder Number	Infuset Reference	Average Flow Rate With SCIg60 Infuser (mL/hr)	Box Qty
FP – 0010004	Infuset – 1850	2,089	25
FP – 0010005	Infuset – 930	876	25
FP – 0010006	Infuset – 820	753	25
FP – 0010007	Infuset – 290	293	25
FP – 0010008	Infuset – 190	189	25
FP – 0010009	Infuset – 650	585	25
FP – 0010010	Infuset – 430	429	25
FP – 0010011	Infuset – 120	122	25
FP – 0010013	Infuset – 45	48	25
FP – 0010014	Infuset – 80	84	25
FP – 0010027	Infuset – 3200	2974	25
FP – 0010028	Infuset – 4000	4084	25
FP – 0010029	Infuset – 4300	4342	25