



Trays Needed:
IM Nail Base Tray
Tibia Nails and Instruments
Ball Tip Guide Rod (Sterile Box)

ORTEMIZ TIBIA NAIL SYSTEM: Setup Guide

Bone Preparation

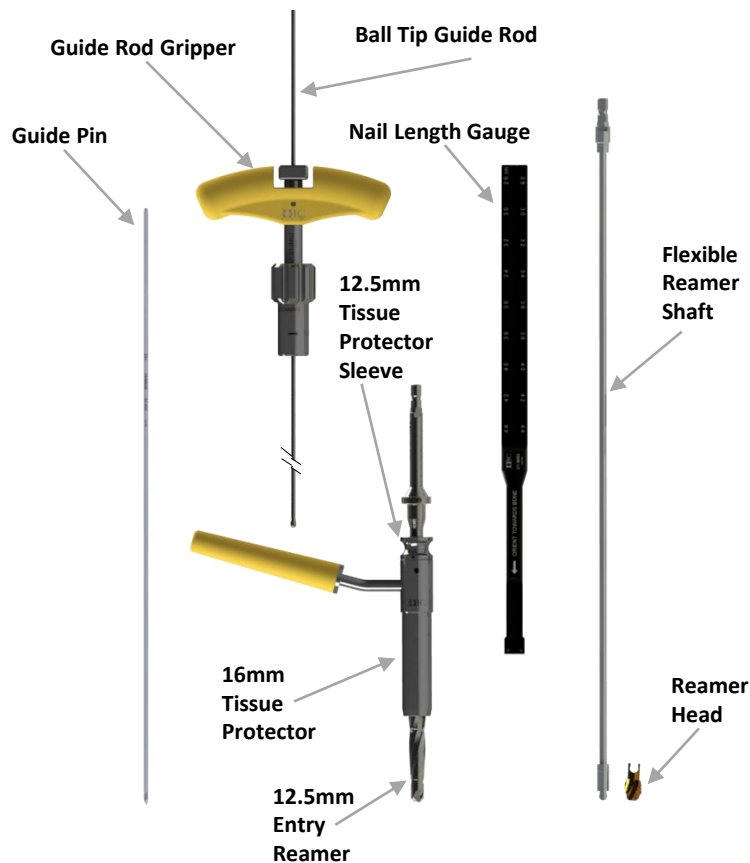

Nail Insertion

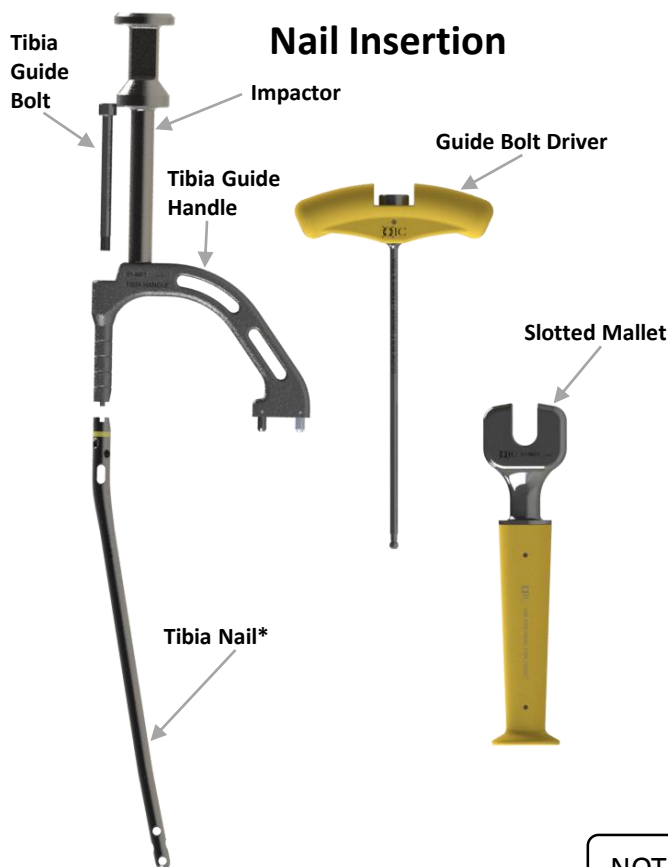


NOTE


*To ensure proper nail orientation, align the "A" on the nail with the "A" on the tibia handle.

Infrapatellar Approach

Proximal Cross Lock Screw Insertion

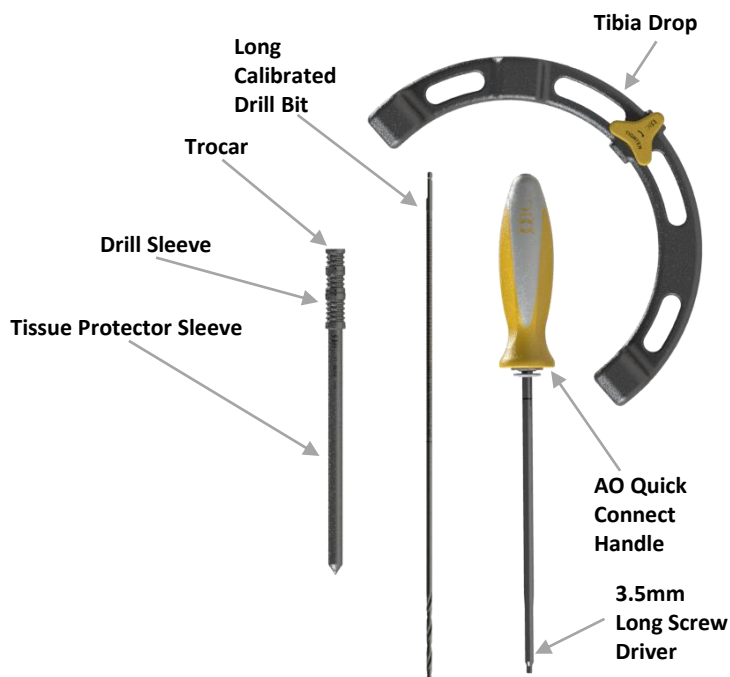

Distal Cross Lock Screw Insertion

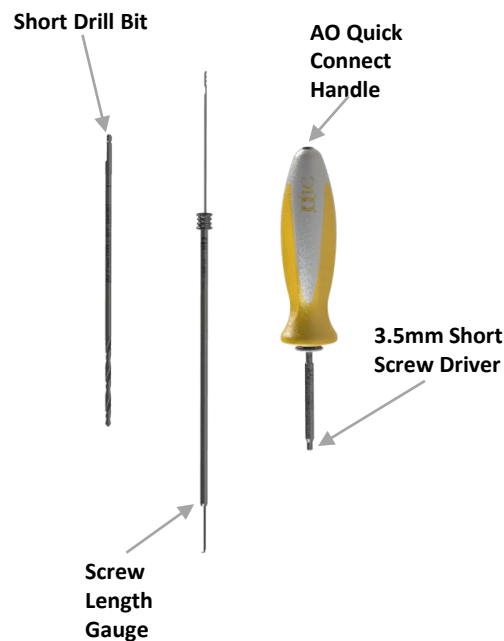


Trays Needed:


IM Nail Base Tray
Tibia Nails and Instruments +
Semi-Extended Instruments
Ball Tip Guide Rod (Sterile Box)

ORTEMIZ TIBIA NAIL SYSTEM: Setup Guide

Bone Preparation

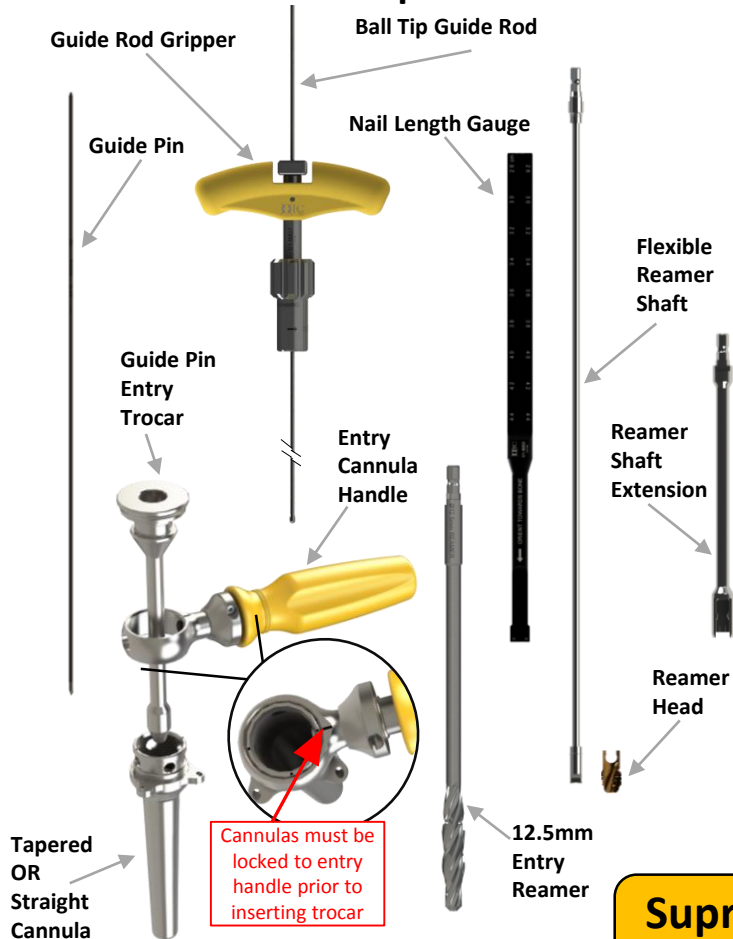

Nail Insertion

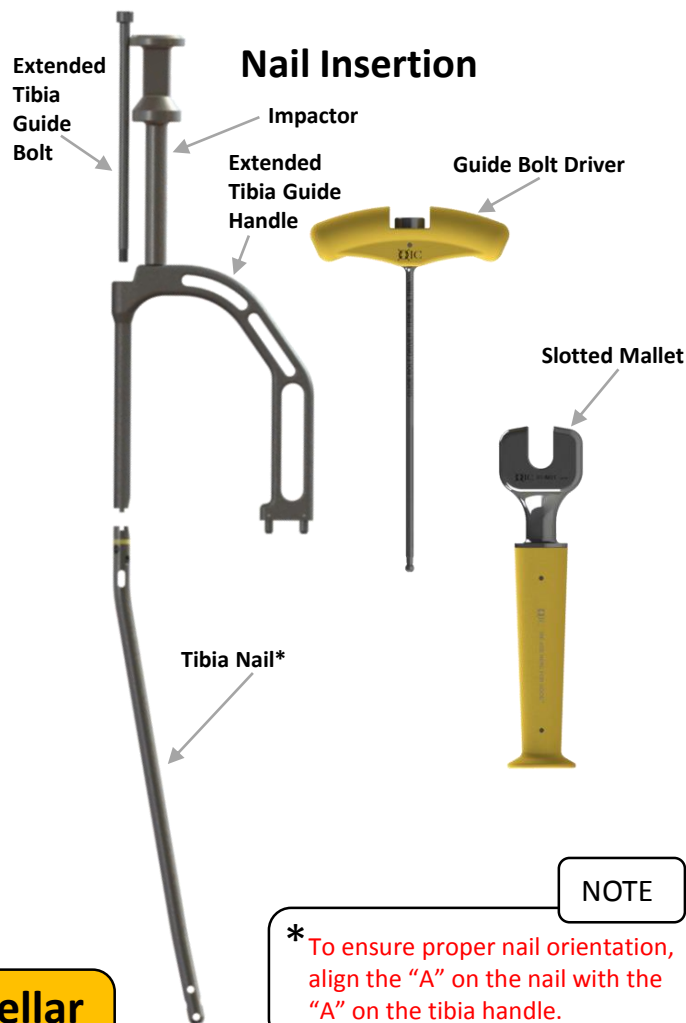


NOTE

*To ensure proper nail orientation, align the "A" on the nail with the "A" on the tibia handle.

Suprapatellar Approach

Proximal Cross Lock Screw Insertion

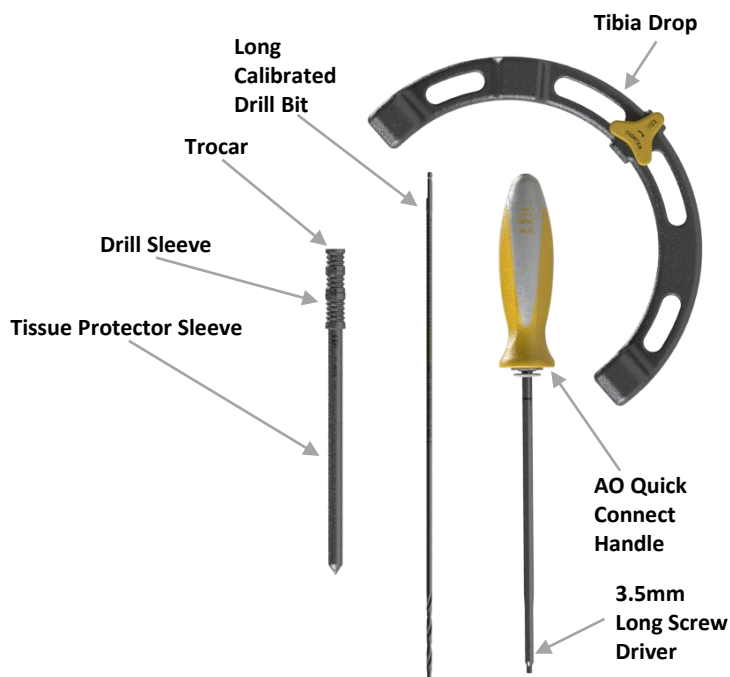

Distal Cross Lock Screw Insertion

