



Uncover the cause
of your gastric
symptoms

Get answers with the
Gastric Alimetry Test



To find out more, visit www.alimetry.com


PM-042 Ver. 0 - November 2024

Symptoms to Solutions

Gastric symptoms such as abdominal pain, bloating, and nausea have a big impact on your life.

It can be hard for doctors to identify the problem, leaving you frustrated and searching for answers.

Fortunately, there's a test that can help, called Gastric Alimetry.

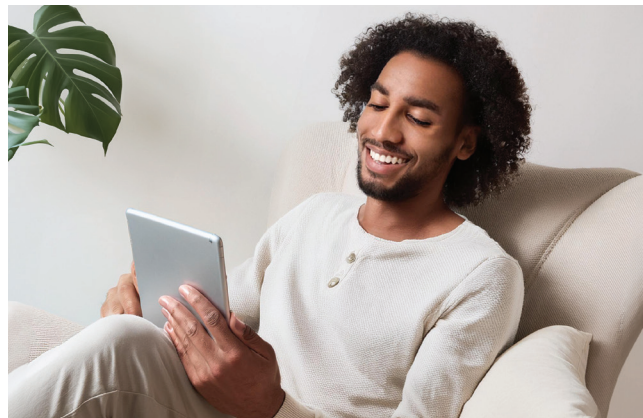

Ask your doctor to refer you for a Gastric Alimetry Test.


The Gastric Alimetry test is available through gastroenterologists in the United States of America, United Kingdom, and New Zealand.

What is the Gastric Alimetry Test?

Gastric Alimetry is a non-invasive device that can help to diagnose functional gastric disorders.

The test provides your doctor with information about your:

- Stomach function
- Symptom experience
- Wellbeing

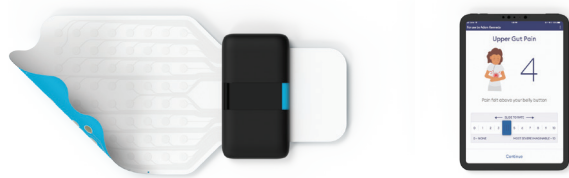

How does it work?

- Similar to your heart, your stomach is made up of muscles.
- The Gastric Alimetry Test uses a sticky patch of sensors and a recording device placed onto the skin over your stomach to read the activity of the stomach muscles.
- A report based on the test provides doctors with information that may relate to certain conditions.

What does the test involve?

1

Wear the device

The sensors placed on your stomach collect signals for 30 minutes while your stomach is empty.

2

Complete the Wellbeing Survey

An optional step to provide a more in-depth overview of your symptoms for your doctor (only available for patients 18 years or older).

3

Consume a standardized meal

Eat a light meal (which can be adjusted to suit dietary restrictions), to see how your stomach responds to food.

4

Rest while it records

Sit comfortably in a recliner chair for 4 hours while the device records. You are free to read, use a device, or go to the toilet.

5

Record your symptoms

Update your symptoms every 15 minutes on the Alimetry iPad App. This captures real-time changes to your symptoms during the test for your clinician to interpret.

6

Get your results

A report will be generated from all the data collected. Your doctor will then share the results with you and use the report to create a treatment plan.

