

MRA Neck W & WO			
Technical Parameters			
Anatomic Coverage:	Aortic Arch to Basilar		
Coil:	NV		
Scanner Preference:	1.5T or 3T		
Contrast:	Pre-injection:		Post-injection:
Scanning Plane:	Axial	Axial (dissection)	Coronal
Slice Thickness:	3mm	4mm	1.2mm
Gap:	-1mm	0.4mm	0mm
Weighting:	2D TOF	T1 FAT SAT	CE 3D TOF
Mode:	FFE	TSE	3D FFE
FAT SAT:	NO	YES	NO
FOV:	20cm	20cm	35cm
MPR:	NO	NO	180 SPIN MIP
Notes:			