

MRA Neck WO		
Technical Parameters		
Anatomic Coverage:	Aortic Arch to Basilar	
Coil:	NV	
Scanner Preference:	1.5T or 3T	
Scanning Plane:	Axial	Axial (for dissection)
Slice Thickness:	3mm	4mm
Gap:	-1mm	0.4mm
Weighting:	2D TOF	T1 FAT SAT
Mode:	FFE	TSE
FAT SAT:	NO	SPIR
FOV:	20cm	20cm
MPR:	NO	YES
Notes:		