

MRI C-Spine W & WO							
Technical Parameters							
Anatomic Coverage:	Foramen magnum to mid T1						
Coil:	NV						
Scanner Preference:	1.5T or 3T						
Contrast:	Pre-injection:				Post-injection:		
Scanning Plane:	Sagittal	Sagittal	Sagittal	Axial	Sagittal	Axial	Coronal
Slice Thickness:	1mm	3mm	3mm	3mm	3mm	3mm	3mm
Gap:	0mm	0.3mm	0.3mm	0.3mm	0.3mm	0mm	0.3mm
Weighting:	T2	T2	T1 FLAIR	T2	T1	T1	T1
Mode:	3D TSE	2D TSE	2D IR-TSE	2D TSE	2D TSE	VIBE/THRIVE	2D TSE
FAT SAT:	NO	DIXON: IN PHASE	NO	NO	DIXON: IN PHASE	YES	DIXON: IN PHASE
FOV:				14cm		15cm	
MPR:	AXIAL 1/0	NO	NO	NO	NO	NO	NO
Notes:							