

MRI Face, Sinus, Skull Base W & WO						
Technical Parameters						
Anatomic Coverage:	Top of frontal sinus to hyoid bone					
Coil:	NV					
Scanner Preference:						
Contrast:	Pre-injection:			Post-injection:		
Scanning Plane:	Sagittal	Axial	Coronal	Axial	Sagittal	Coronal
Slice Thickness:	0.7mm	3mm	2.5mm	3mm	0.7mm	2mm
Gap:	0mm	0.3mm	0.25mm	0.3mm	0mm	0.25mm
Weighting:	T1	DWI	T2	T2	T1	T1
Mode:	3D TSE	2D EPI	2D TSE	2D TSE	3D TSE	2D TSE
FAT SAT:	NO	SPIR	DIXON: IN PHASE	DIXON: IN PHASE	YES	DIXON: IN PHASE
FOV:	18cm	22cm	18cm	18cm	18cm	18cm
MPR:	NO	NO	NO	NO	AXIAL, COR	NO
Notes:						