

MRI T-Spine W & WO

Technical Parameters

Anatomic Coverage:	C6-C7 to L1-L2					
Coil:	Torso and/or NV					
Scanner Preference:	1.5T or 3T					
Contrast:	Pre-injection:			Post-injection:		
Scanning Plane:	Sagittal	Sagittal	Sagittal	Sagittal	Axial	Coronal
Slice Thickness:	1mm	3mm	3mm	3mm	3mm	3mm
Gap:	0mm	0.3mm	0.3mm	0.3mm	0mm	0.3mm
Weighting:	T2	T2	T1 FLAIR	T1	T1	T1
Mode:	3D TSE	2D TSE	2D IR-TSE	2D TSE	VIBE/THRIVE	2D TSE
FAT SAT:	NO	DIXON: IN PHASE	NO	DIXON: IN PHASE	YES	DIXON: IN PHASE
FOV:					15cm	
MPR:	AXIAL 1/0	NO	NO	NO	NO	NO
Notes:						