

MRV Brain W & WO			
Technical Parameters			
Anatomic Coverage:	Vertex to skull base; frontal through occipital, skin to skin		
Coil:	Head		
Scanner Preference:	1.5T or 3T		
Contrast:	Pre-injection:		Post-injection:
Scanning Plane:	Axial	Coronal	Sagittal
Slice Thickness:	2mm	2mm	~1mm
Gap:	-0.5mm	-0.5mm	~0mm
Weighting:	T1	T1	T1
Mode:	M2D FFE	M2D FFE	3D CE-MRV
FAT SAT:	NO	NO	NO
FOV:	23cm	23cm	24cm
MPR:	NO	NO	NO
Notes:			