

MRI Forefoot WO						
Technical Parameters						
Anatomic Coverage:	Midfoot through toes					
Coil:	Foot or Ankle coil					
Scanner Preference:	1.5T or 3T					
Scanning Plane:	Axial	Axial	Sagittal	Sagittal	Coronal	Coronal
Slice Thickness:	3mm	3mm	3mm	3mm	3mm	3mm
Gap:	0.5mm	0.5mm	0.5mm	0.5mm	0.5mm	0.5mm
Weighting:	PD	PD FS	STIR	T1	PD	PD FS
Mode:	TSE	TSE	TSE	TSE	TSE	TSE
FAT SAT:	NO	YES	YES	NO	NO	YES
FOV:	12cm	12cm	12cm	12cm	12cm	12cm
MPR:	NO	NO	NO	NO	NO	NO
Notes:						

