

MRI Hip Arthrogram				MRI Hip WO (for reference)					
Technical Parameters				Technical Parameters					
Anatomic Coverage:	Above acetabulum to lesser trochanters			Anatomic Coverage:	Above acetabulum to lesser trochanters				
Coil:	Torso			Coil:	Torso				
Scanner Preference:	1.5T or 3T			Scanner Preference:	1.5T or 3T				
Contrast:	Post-injection:			Scanning Plane:	Axial	Coronal	Coronal	Coronal	Sagittal
				Slice Thickness:	4mm	5mm	4mm	4mm	4mm
Slice Thickness:	4mm	4mm	4mm	Gap:	1mm	1mm	1mm	1mm	1mm
Gap:	1mm	1mm	1mm	Weighting:	PD FS	STIR	T1	PD FS	PD FS
Weighting:	T1 FS	T1 FS	T1 FS	Mode:	TSE	TSE	TSE	TSE	TSE
Mode:	TSE	TSE	TSE	FAT SAT:	YES	YES	NO	YES	YES
FAT SAT:	YES	YES	YES	FOV:	18cm	Large FOV to include entire pelvis 30-40cm	18cm	18cm	18cm
FOV:	18cm	18cm	18cm	MPR:	NO	NO	NO	NO	NO
MPR:	NO	NO	NO	Notes:					
Notes:									

