

MRA Mesenteric Vessels											
Technical Parameters											
Anatomic Coverage:	AP coverage behind aorta through entire kidneys										
Coil:	None										
Scanner Preference:											
Contrast:	Pre-injection:							Post-injection:			
	Scanning Plane:	Coronal	Coronal	Axial	Axial	Axial	Axial	Coronal	Coronal	Coronal	Axial
Slice Thickness:	4mm	4mm	6mm	6mm	3mm	3mm	1.1mm	1.1mm	1.1mm	3mm	3mm
Gap:	0mm	0mm	0mm	0mm	0mm	0mm	0mm	0mm	0mm	0mm	0mm
Weighting:	T2	T2	T2	T2	T1	T1	FLASH	FLASH	FLASH	T1	T1
Mode:	HASTE BH	TRUFI BH	HASTE BH	TRUFI BH	VIBE DIXON ABD	VIBE DIXON BH PELVIS	3D BH PRE	3D DYNAMIC BH PHASE 1	3D DYNAMIC BH PHASE 2	VIBE DIXON BH ABD	VIBE DIXON BH PELVIS
FAT SAT:	NO	NO	NO	NO	YES	YES	YES	YES	YES	YES	YES
FOV:	Fit to Patient	Fit to Patient	Fit to Patient	Fit to Patient	Fit to Patient	Fit to Patient	Fit to Patient	Fit to Patient	Fit to Patient	Fit to Patient	Fit to Patient
MPR:	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO	NO
Notes:											