

MRI Abdomen W & WO								
Technical Parameters								
Anatomic Coverage:	Fit to Patient							
Coil:	Torso							
Scanner Preference:	1.5T							
Contrast:	Pre-injection:						Post-injection: trigger when bolus reaches SMA	
Scanning Plane:	Cor Abd	Axial Abd	Axial Abd	Axial Abd	Axial Abd	Axial Abd	Axial Abd	Axial Abd (4 min delay)
Slice Thickness:	7mm	7mm	7mm	5mm	7mm	5mm	5mm	5mm
Gap:	1mm	1mm	1mm	1mm	1mm	1mm	1mm	1mm
Weighting:	T2	T1	T2	T2	T2	T1	T1	T1
Mode:	SS BH	Dual Echo SPGR BH	DWI	S5	TSE RT	3D mDixon pre BH	3D mDixon x3 phases BH	3D mDixon BH
FAT SAT:	NO	NO	SPIR	NO	SPAIR	YES	YES	YES
FOV:	Fit to Patient	Fit to Patient	Fit to Patient	20-24mm	Fit to Patient	Fit to Patient	Fit to Patient	Fit to Patient
MPR:	NO	NO	NO	NO	NO	NO	NO	NO
Notes:								