

MRI Prostate WO					
Technical Parameters					
Anatomic Coverage:	Prostate to seminal vesicles				
Coil:	Endorectal coil if 1.5T scanner; no coil if 3T scanner				
Scanner Preference:	3T				
Scanning Plane:	Axial	Sagittal	Axial	Coronal	Axial
Slice Thickness:	5mm	3mm	3mm	3mm	3mm
Gap:	0.5mm	0mm	0mm	0mm	0.3mm
Weighting:	T1	T2	T2 HR	T2	DWI
Mode:	TSE	TSE	TSE	TSE	EPI
FAT SAT:	NO	NO	NO	NO	NO
FOV:	240x300	180x180	180x180	180x180	180x180
Notes:					