

CT Abdomen Pelvis Without Contrast

Technical Parameters

kVp:	Siemens: Care kV (120 reference) GE: Auto Prescription			
mAs:	Dose Modulation System (ex CARE Dose 4D, smart mA)			
Pitch:	<1 preferred			
Collimation:	Siemens (128x0.6) GE (40mm or 80mm)			
Scan Direction:	Craniocaudal			
Rotation Time:	1s or less			
Anatomic Coverage:	Scan should cover from just above the diaphragm through the lesser trochanters Have patient take a breath in and hold it for the scan.			
Reconstruction Parameters:	Soft Tissue Ax	Coronal	Sagittal	Axial Thins
Slice Thickness:	5mm	2.5mm	2.5mm	2.5mm
Reconstruction Interval:	5mm	2.5mm	2.5mm	2.5mm
Reconstruction FOV:	Smallest area to include relevant anatomy	Smallest area to include relevant anatomy	Smallest area to include relevant anatomy	Smallest area to include relevant anatomy
Window	Siemens (Mediastinum) GE (WW 350 WL 50)	Siemens (Mediastinum) GE (WW 350 WL 50)	Siemens (Mediastinum) GE (WW 350 WL 50)	Siemens (Mediastinum) GE (WW 350 WL 50)
Kernel/Algorithm:	Siemens (Br 36) GE (Standard)	Siemens (Br 36) GE (Standard)	Siemens (Br 36) GE (Standard)	Siemens (Br 36) GE (Standard)

CT Abdomen Pelvis With Contrast					
Technical Parameters					
kVp:	Siemens: Care kV (120 reference) GE: Auto Prescription				
mAs:	Dose Modulation System (ex CAREDose 4D, smart mA)				
Pitch:	<1 preferred				
Collimation:	Siemens (128x0.6) GE (40mm or 80mm)				
Scan Direction:	Craniocaudal				
Rotation Time:	1s or less				
Anatomic Coverage:	Scan should cover from just above the diaphragm through the lesser trochanters Have patient take a breath in and hold it for the scan.				
Contrast:	75mL Omni300/Iso300 3mL/s 70 second delay after injection before scanning				
Reconstruction Parameters:	Soft Tissue Ax	Lung	Coronal	Sagittal	Axial Thins
Slice Thickness:	5mm	5mm	2.5mm	2.5mm	2.5mm
Reconstruction Interval:	5mm	5mm	2.5mm	2.5mm	2.5mm
Reconstruction FOV:	Smallest area to include relevant anatomy	Smallest area to include relevant anatomy	Smallest area to include relevant anatomy	Smallest area to include relevant anatomy	Smallest area to include relevant anatomy
Window	Siemens (Mediastinum) GE (WW 350)	Siemens (Lung) GE (WW 1500 WL -600)	Siemens (Mediastinum) GE (WW 350)	Siemens (Mediastinum) GE (WW 350)	Siemens (Mediastinum) GE (WW 350)
Kernel/Algorithm:	Siemens (Br 36)	Siemens (Br 69)	Siemens (Br 36)	Siemens (Br 36)	Siemens (Br