

How to use eye drop glasses


Read carefully before use.




Make sure the eye drops are still good to use. The drops will stay good for up to a month.




Wash your hands and the eye drop glasses.

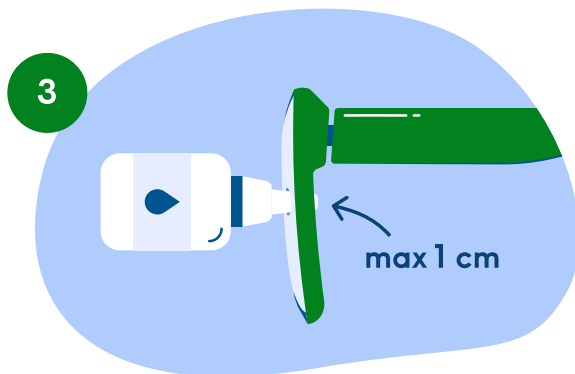

Place the dosing bottle in the hole that fits best. The bottle should not stick out more than 1 cm.

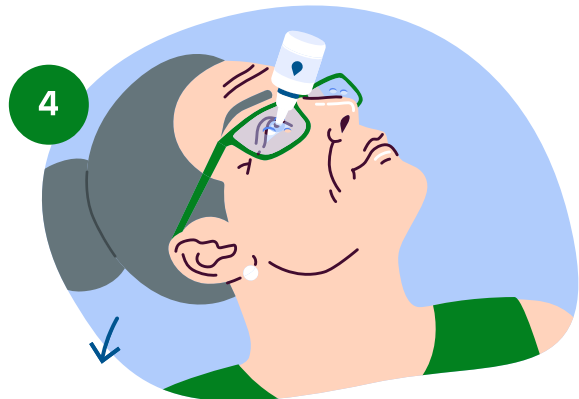

Put the glasses on and lean your head back as far as possible. Or lie down.

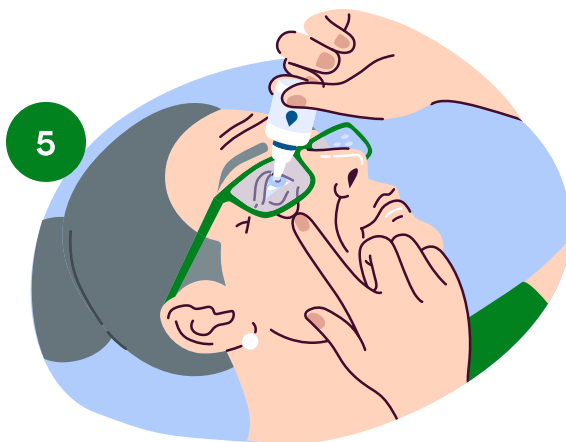

Pull your lower eyelid down and keep your eye open. Squeeze the bottle.



Take off the glasses. Press the inner corner of the eye for 1 minute. Clean the glasses.

Scan this QR code to watch the instruction video

